TGR-Y MR 001-5

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STARTING PROCEDURE

KOMPRESOR SOFT WIND VI

AIR PUMP SOFT WIND VI

PL Kompresor SOFT WIND VI

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z Air pump SOFT WIND VI

Starting procedure

FOR THE ASSEMBLY AND SET-UP DESCRIBED BELOW, THE USER MAY REQUIRE THE HELP OF ANOTHER PERSON. YOU SHOULD ASK FOR HELP FROM A PERSON WITH PROPER SKILLS WHO SHOULD FOLLOW INFORMATION INCLUDED IN THE INSTRUCTIONS.

THE MATTRESS SHOULD NOT BE USED IF IT IS NOT FULLY INFLATED.

1. Place the mattress on the bed directly on top of the regular mattress. The flaps at each end of the anti-decubitus mattress should be tucked under the regular mattress to limit movement of one mattress relative to the other. Next, cover the mattress with the cover included in the set. It is recommended that the mattress be secured with a hygienic vapour-permeable mattress pad or cotton sheet in order to prevent it from being soiled.
2. The hook of the pump should be opened to the desired width and hung on the bed frame.
3. The pump and the mattress should be connected with air hoses so that they are not kinked or trapped under the mattress. Make sure that the hoses are properly fastened and do not come out from the ports.
4. Plug in the pump using the power cord, taking extra care while doing this.
5. To start the pump, press the ON button on the control panel. The anti-decubitus mattress will then start to inflate.
6. Set the pressure dial at the maximum value. The preparation time (the first inflation) prior to use takes 24 minutes (two full cycles).

YOU MUST WAIT 24 MINUTES FROM STARTING THE MATTRESS BEFORE USE.

7. Following complete inflation, the pressure dial should be set at a value appropriate to the patient's weight. The anti-decubitus mattress with air pump will then be ready to use.

THE PRESSURE SHOULD BE APPROPRIATE FOR THE USER'S INDIVIDUAL NEEDS DETERMINED BY HIS/HER GENERAL CONDITION AND COMPLAINTS ASSOCIATED WITH THEIR CONDITION. TO OBTAIN THIS INFORMATION YOU SHOULD CONTACT YOUR DOCTOR.

THE AIR PUMP OPERATES BY ALTERNATELY PUMPING AIR TO MATTRESS AIR CELLS AND FOR THIS REASON NEEDS TO BE ON ALL THE TIME. THE OFF BUTTON ON THE CONTROL PANEL IS ONLY FOR TURNING OFF THE PUMP WHEN THE MATTRESS IS NOT IN USE. BEFORE USING THE MATTRESS AGAIN, THE STARTING PROCEDURE MUST BE PERFORMED (SEE STEPS 6 AND 7)

TECHNICAL SPECIFICATIONS:

- Air pump model: SOFT WIND VI
- Air pump weight: 1 kg
- Cycle duration: 12 minutes
- Pressure range: 40-110 mmHg
- Flow: 6-7l/min
- Power: AC 220V 50/60 Hz 0,1A
- Fuse: T1A 250V
- Wattage: 10 W
- Operating environment:
 - Temperature: + 10°C – 40°C
 - Humidity: 10% - 90%

SAFETY RULES

IN PATIENTS WITH A PACEMAKER, THE PUMP SHOULD BE PLACED AS FAR FROM THE CHEST AS POSSIBLE.

ONE MATTRESS SHOULD ONLY BE USED FOR ONE PATIENT.

DO NOT EXCEED THE MAXIMUM WEIGHT CAPACITY (140 KG).

NO REHABILITATION PROCEDURES SHOULD BE PERFORMED ON THE MATTRESS.

OTHER PEOPLE SHOULD NOT SIT ON THE MATTRESS.

THE MATTRESS MAY ONLY BE USED ON FLAT SURFACES. THE MATTRESS SHOULD BE PROTECTED AGAINST SHARP OBJECTS AND SHOULD NOT BE LAID DIRECTLY ON BED SPRINGS OR BED FRAMES WITH SHARP EDGES.

THE AIR PUMP SHOULD NOT BE USED IF THE POWER CORD OR PLUG IS DAMAGED, OR IN CASE OF APPARENT PRODUCT FAILURE, FOR EXAMPLE, INSUFFICIENT PUMPING FORCE. THE AIR PUMP SHOULD NOT BE CONNECTED TO DAMAGED EXTENSION CORDS OR ELECTRIC OUTLETS.

A WORN OUT OR DAMAGED AIR PUMP SHOULD BE TAKEN TO AN ELECTRICAL OR ELECTRONIC RECYCLING CENTRE. DO NOT DISPOSE THE AIR PUMP WITH OTHER MUNICIPAL WASTE.

More detailed information concerning the equipment can be found in the user manual

**TIMAGO INTERNATIONAL GROUP
OSSOWSKI SP. K.
UL. KARPACKA 24/12**

43-316 Bielsko-Biała, Polska
T.: +48 33 499 50 00
F.: +48 33 499 50 11

TIMAGO.COM