**THUASNE**

# Ortel C2 Plus

fr	Collier cervical semi-rigide.....	6
en	Semi-rigid cervical collar .....	7
de	Halbfeste Halskrause .....	8
nl	Halfstijve nekbrace .....	9
it	Collare cervicale semi-rigido .....	11
es	Collarín cervical semirrígido .....	12
pt	Colar cervical semirrígido.....	13
da	Halvstiv nakkekrave.....	15
fi	Puolijäykkä kaulatuki.....	16
sv	Halstyv nackkrage .....	17
el	Αυθενικό κολάρο ημίσκληρο.....	18
cs	Polotuhý krční límec.....	20
pl	Kołnierz pół-sztywny do unieruchomienia szyi.....	21
lv	Dalēji stingra kakla ortoze.....	22
lt	Pusiau standus kaklo įtvaras .....	24
et	Pooljäik emakakaela krae.....	25
sl	Opornica za vrat, poltoga.....	26
sk	Polovystužený krčný golier.....	27
hu	Félkemény nyakmerevítő .....	29
bg	Полутвърда цервикална яка .....	30
ro	Guler cervical semirigid .....	31
ru	Полужесткий шейный воротник.....	33
hr	Polutvrda ortoza za vratnu kralježnicu .....	34
zh	半硬质颈圈.....	35
ar	طوق للعنق صلب جزئياً.....	39




	a	b		
<b>0</b>	24 - 29 cm	6 cm	9 cm	11 cm
<b>1</b>	28 - 33 cm			
<b>2</b>	34 - 39 cm			
<b>3</b>	40 - 46 cm	-		

<b>fr</b>	(a) Tour de cou (b) Hauteur de cou	<b>lv</b>	(a) Kakla apkārtmērs (b) Kakla augstums
<b>en</b>	(a) Neck circumference (b) Neck height	<b>it</b>	(a) Kaklo apimtis (a) Kaklo aukštis
<b>de</b>	(a) Halsumfang (b) Halshöhe	<b>et</b>	(a) Kaela ümbermõõt (b) Kaela pikkus
<b>nl</b>	(a) Omtrek van de nek (b) Hoogte van de nek	<b>si</b>	(a) Obseg vratu (b) Dolžina vratu
<b>it</b>	(a) Circonferenza del collo (b) Altezza del collo	<b>sk</b>	(a) Obvod krku (b) Výška krku
<b>es</b>	(a) Circunferencia del cuello (b) Altura del cuello	<b>hu</b>	(a) Nyak kerülete (b) Nyak magassága
<b>pt</b>	(a) Perímetro do pescoço (b) Altura do pescoço	<b>bg</b>	(a) Размер на яката (b) Височина на яката
<b>da</b>	(a) Halsens omkreds (b) Halsens højde	<b>ro</b>	(a) Circumferința gâtului (b) Înălțimea gâtului
<b>fi</b>	(a) Kaulan ympärys (b) Kaulan korkeus	<b>ru</b>	(a) Обхват шеи (b) Высота шеи
<b>sv</b>	(a) Runt halsen (b) I höjd med halsen	<b>hr</b>	(a) Opseg vrata (b) Visina vrata
<b>el</b>	(a) Περίμετρος λαιμού (b) Ύψος λαιμού	<b>zh</b>	( a ) 颈围 ( b ) 颈高
<b>cs</b>	(a) Obvod krku (b) Výška krku	<b>ar</b>	محيط العنق (a) ارتفاع العنق (b)
<b>pl</b>	(a) Obwód szyi (a) Wysokość szyi		

<b>fr</b>	Stabilisation
<b>en</b>	Stabilisation
<b>de</b>	Stabilisierung
<b>nl</b>	Stabilisatie
<b>it</b>	Stabilizzazione
<b>es</b>	Estabilización
<b>pt</b>	Estabilização
<b>da</b>	Stabilisering
<b>fi</b>	Stabiloi
<b>sv</b>	Stabilisering
<b>el</b>	Σταθεροποίηση
<b>cs</b>	Stabilizace
<b>pl</b>	Stabilizacja
<b>lv</b>	Stabilizēšana
<b>lt</b>	Stabilizavimas
<b>et</b>	Stabiliseerimine
<b>si</b>	Stabiliziranje
<b>sk</b>	Stabilizácia
<b>hu</b>	Stabilizálás
<b>bg</b>	Стабилизация
<b>ro</b>	Stabilizare
<b>ru</b>	Стабилизация
<b>hr</b>	Stabilizacija
<b>zh</b>	稳定
<b>ar</b>	الاستيـت

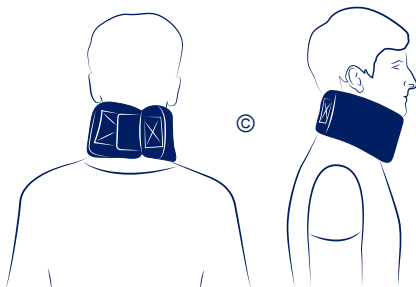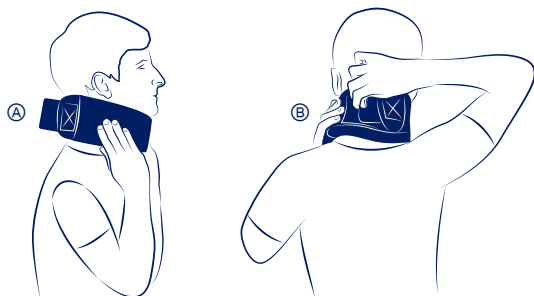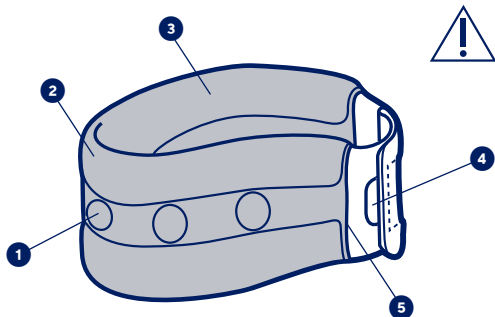**MD**

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## SEMI-RIGID CERVICAL COLLAR

### Description/Destination

Light, aerated semi-rigid structure. ①  
Anatomically-shaped support foam. ②  
Rounded edges for appropriate comfort. ③  
Precise self-fastening adjustment. ④  
Convenient removable cover. ⑤

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Cotton - polyamide - elastane - polyethylene - acrylic polymers - polyurethane.

### Properties/Mode of action

Stabilization of the cervical spine.

### Indications

Cervical arthritis.  
Cervical syndrome.  
Dropped head syndrome.  
Mild whiplash/mild sprain.  
Cervicobrachial neuralgia.  
Rehabilitation after wearing a rigid cervical collar.  
Torticolis (stiff neck).

### Contraindications

Do not apply the product in direct contact with broken skin.  
Do not use in the event of known allergy to any of the components.  
Do not use in the event of traumatic injuries: severe sprain, fracture.  
Do not use for prolonged wear without medical supervision.  
Do not use the product if the diagnosis has not been confirmed.

### Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.  
In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.  
Choose the appropriate size to fit the patient, referring to the size chart.  
Verify the product's integrity before every use.  
It is recommended that a healthcare professional supervises the first application.  
For hygiene and performance reasons, do not re-use the product for another patient.  
Do not use the device if it is damaged.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without excessive compression.

Do not wear the product in a medical imaging machine.

Do not wear the product while sleeping.

#### **Undesirable side-effects**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

In the event of long-term wear of the product, possible secondary risks related to muscle atrophy.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### **Instructions for use/Application**

Open the self-fastener.

Place the collar around the neck ⊕.

The chin should rest on the chin support.

Close the self-fastener at the nape of the neck ⊕.

Ensure the strap is not fastened too tight ⊕.

#### **Care**

See product label. Close the self-fastening tabs before washing. Hand wash.

Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

#### **Storage**

Store at room temperature, preferably in the original packaging.

#### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2029002 (2022-03)